


## Easyboot Rx Size Chart

Size	Width in inches	Length in inches	Size	Width in mm	Length in mm
00	3 13/16" - 4 3/8"	4 3/16" - 4 5/8"	00	98-110mm	106-117mm
0	4 3/8" - 4 9/16"	4 5/8" - 4 15/16"	0	111-116mm	118-124mm
1	4 9/16" - 4 7/8"	4 15/16" - 5 1/8"	1	117-123mm	125-130mm
2	4 7/8" - 5 1/8"	5 1/8" - 5 9/16"	2	124-130mm	131-140mm
3	5 1/8" - 5 1/2"	5 9/16" - 5 15/16"	3	131-139mm	141-150mm
4	5 1/2" - 5 15/16"	5 15/16" - 6 7/16"	4	140-150mm	151-163mm
5	5 15/16" - 6 7/16"	6 7/16" - 6 7/8"	5	151-163mm	164-174mm
6	6 7/16" - 6 7/8"	6 7/8" - 7 7/16"	6	164-175mm	175-188mm
7	6 7/8" - 7 9/16"	7 7/16" - 7 3/4"	7	176-192mm	189-195mm

### How to measure for the Easyboot Rx:

After a fresh trim, measure the width of the hoof across the bottom at the widest point.

1. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.
2. Compare your measurement with the size chart above.
3. Ideally, the length and width measurement will fit into the same size. If the length and width measurements indicate different sizes, select the larger size. If the width and length measurements are different by more than one size, your selected boot style is not recommended. Be sure to check your measurements against an alternative boot in the EasyCare line—another style may accommodate your horse's hoof shape better.

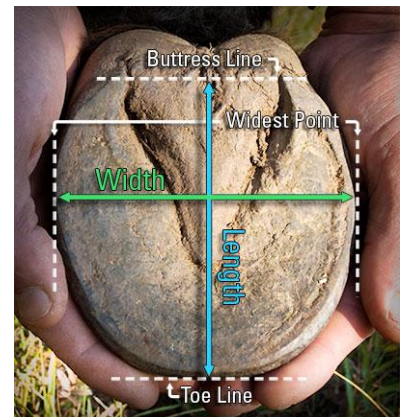

*Be sure to size up if you plan on using bandages or 12mm Comfort Pads with the Easyboot Rx.*

*lan on using bandages or 12mm Comfort Pads with the Easyboot Rx.*

To order go to [www.easycaredownunder.com.au](http://www.easycaredownunder.com.au)  
or phone 02 4455-2440

