




Easyboot Glove Wide

**The new WIDE GLOVE is here!**  
**Plus the WIDE Glue on Shells.**

Wide Glove & Wide Glue On Shells Sizing		
Size:	Width in mm	Length in mm
0 wide	114 – 117 mm	114 – 117 mm
0.5 wide	118 – 121 mm	118 – 121 mm
1 wide	122 – 125 mm	122 – 125 mm
1.5 wide	126 – 129 mm	126 – 129 mm
2 wide	130 – 133 mm	130 – 133 mm
2.5 wide	134 – 137 mm	134 – 137 mm
3 wide	138 – 141 mm	138 – 141 mm

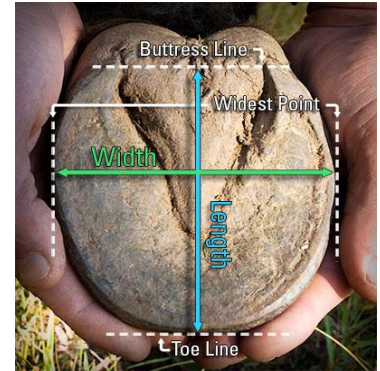

**Success of the Easyboot Glove and Easyboot Glue-On are determined by accurate measurements and a snug fit.**

How to measure for the Easyboot Glove:

1. After a fresh trim, measure the width of the hoof across the bottom at the widest point.
2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.
3. Compare your measurements with the appropriate size chart. Ideally, the length and width measurements should fit into the same size on the chart.
4. Because this boot needs to fit very snug, if you are a size apart we recommend that you order a **fit kit**. Be sure to check the requirements and measurements against an alternative boot in the EasyCare line—another style may accommodate your horse's hoof shape better

## What is a fit kit?

By lodging a refundable deposit with your boot supplier they can offer you a “fit kit” of shells to help you get a perfect fit with The WIDE Glove Boot. These shells are the base portion of the Glove Boot (the Glue on Shell). Using these allows you to fine tune your fit. Glove boots more than any other style rely on being a snug fit on the hoof and also up the walls. They are also more stretchy than any other style so often with a fit kit you can assess if a fit can still be achieved if the hoof is just outside the sizing chart measurements.

The new Easyboot Glove and Easyboot Glue-On horse boots represent a revolution in hoof boot design. They are lightweight and simple, and fit the hoof seamlessly.

Because both new boots have no hardware or fastening systems, achieving a good fit is more important than ever. And because proper fit is so vital, we decided it would be best to offer these boots in half sizes.

Each shell has only four millimeters of width difference between the next shell size. For reference, 4mm is the width of 3 dimes. This tight gap between sizes helps ensure a snug fit, but it also means that there is little room for error in measuring and determining the proper size of boot.

If you are new to boots or have concerns about fitting the WIDE Glove boots then contact Easycare Down Under who have professionals on staff to help you with all your hoof care issues. [www.easycaredownunder.com.au](http://www.easycaredownunder.com.au)